

# InnoSpire Go

portable mesh nebuliser



## Reduces treatment time by 25%\*

The easy-to-use and virtually silent Philips InnoSpire Go has been specifically designed to shorten nebuliser treatment times, giving your patients more time for life.

Utilizing established, clinically proven vibrating mesh technology adopted by hospitals worldwide, InnoSpire Go delivers all commonly prescribed respiratory medications.



As easy 1,2,3...



- 1. Fill**  
Lift the green lid, pour in the medication and the InnoSpire Go is ready to use.



- 2. Treat**  
Press the on button and the InnoSpire Go will begin to nebulise.



- 3. Clean**  
Remove the mouthpiece and quickly rinse it in warm water.

\* Versus predecessor Aeroneb Go.

InnoSpire Go provides an easy therapy experience, which helps patients administer therapy correctly.



**Treatment in 4 mins\*\***

Efficient aerosol therapy with audible and visible indicators that signal therapy completion - before automatically turning off.

**Virtually silent**

A small, portable and discreet therapy device, allowing your patients to take their therapy when and where they need it.

**Built-in battery**

Rechargeable, long life, built-in battery that provides up to 30 treatments between each charge.

**Simple, only two parts**

The easily detachable mouthpiece makes the InnoSpire Go simple to use, clean and maintain, encouraging greater adherence.\*\*\*

**InnoSpire Go aerosol data**

	Salbutamol 2.5 ml	Ipratropium bromide 2 ml	Budesonide 2 ml
MMAD $\mu\text{m}$	3.99	3.93	4.84
Respirable fraction % particles < 5 $\mu\text{m}$	64.4%	65.3%	52.1%
Treatment time (min:sec)	4:18	3:44	3:34
Residual volume ml	0.3	0.2	0.3

**Technical specifications**

Battery	Lithium ion
Dimensions	7.0 cm x 4.5 cm x 13.5 cm
Weight	111 g
Warranty	Handset 2 years Mouthpiece 1 year



\*\* Using 2.5ml salbutamol

\*\*\* A mask is included for children that find it difficult to use the mouthpiece.

For more information please contact us at:

Air Liquide Healthcare, Stratus House, Unit 1 IDA College Park, Blanchardstown Road North, Blanchardstown, Dublin 15, D15 PEC4.

Tel: +353 (0)1 809 1800 Freephone: 1800 24 02 02 (ROI only) 0800 328 5875 (NI only)

Fax: +353 (0)1 829 3966 Email: healthie@airliquide.ie Website: www.airliquidehealthcare.ie