

# InnoSpire Mini

so your patient can enjoy life



The Philips InnoSpire Mini compressor nebuliser system allows patients to effectively nebulise most commonly prescribed aerosol medications whenever they want and wherever they may be.

**InnoSpire Mini is part of the Philips InnoSpire family of compressor nebuliser systems that provides reliable drug delivery to patients.**

An ideal traveling companion, InnoSpire Mini's compact size allows for easy transport while a breath-enhanced reusable nebuliser makes it easy to use.

## Proven nebuliser technology

Coupled with the SideStream Plus reusable handset, the InnoSpire Mini provides highly efficient aerosol therapy in fast treatment times.

## Key features

- Matched with SideStream Plus
- Compact and lightweight
- Battery powered (optional)
- Up to 90 min autonomy when battery powered
- Multiple charging options





## Reliable aerosol delivery

**SideStream Plus has been designed to draw in more air, via an active venturi system, resulting in fast drug delivery and short treatment times.**

SideStream Plus' unique diamond jet design helps minimize wear from frequent use and aids consistent delivery of a high quality aerosol.

## Giving your patient choice

The InnoSpire Mini can be powered by the traditional wall outlet or by an optional rechargeable lithium ion battery, which can operate the compressor for up to 90 minutes between charges. The InnoSpire Mini can also be powered by a 12V DC car adapter while on the move; its portability enables patients to take their treatment virtually anytime and anywhere, at their convenience.

The versatile InnoSpire Mini is also compliant with selected FAA (US Federal Aviation Administration) environmental standards\*\*\* for airborne equipment.

Product description	Part number
<b>Ordering information</b>	
InnoSpire Mini without battery – EU plug** (with user guide†)	<b>1109291</b>
InnoSpire Mini with battery – EU plug** (with user guide†)	<b>1109292</b>
InnoSpire Mini with battery – EU plug** (with user guide††)	<b>1109438</b>
InnoSpire Mini with battery – EU plug** (with user guide††)	<b>1109294</b>
InnoSpire Mini with battery – UK plug** (with user guide††)	<b>1111453</b>
<b>Accessories/replacement parts</b>	
InnoSpire Mini replacement air filter pack	<b>1109488</b>
SideStream Plus with mouthpiece reusable kit	<b>1092001</b>

- \* Can be fully recharged in approximately 6 hours
- \*\* Including rechargeable battery (except for PN 1109291), 12V DC car adapter, AC adapter, unit carrying case and (4) spare filters
- \*\*\* DO-160G standards for airborne equipment and the international basic safety and essential performance standard IEC 60601-1 third edition
- † Languages: EN, BG, CZ, EL, ES, HR, ID, IT, HU, PT, RO, SK, SL, SR, TR, and KO
- †† Languages: EN, AR, DA, DE, EE, FI, FR, IL, KZ, LT, LV, NL, NO, PL, RU, SV, and TH

## SideStream Plus

Equipped with a low resistance valve to deliver fast treatments with less waste.



## Valve opens

Significantly less pressure is needed to open the inspiratory valve, making it ideal for weaker breathers.



## Filter

Our optional filter accessory also helps to reduce the unnecessary release of medication in ambient air, especially when nebulising antibiotics.



## Performance data

MMAD	4.77 µm
Fine particle fraction	51.44%
Treatment time (2.5ml charge)	6.88 min

## Technical specifications

Compressor max. pressure	207kPa (30psi)
Average flow rate	3.4LPM @ 12.7psi
Weight	Compressor: 420g Battery: 172g
Sound level	50 ± 4 dBA
Operating temperature	5°C to 40°C
Operating humidity	10% RH to 95% RH



### For more information please contact us at:

Air Liquide Healthcare, Stratus House, Unit 1 IDA College Park, Blanchardstown Road North, Blanchardstown, Dublin 15, D15 PEC4.  
Tel: +353 (0)1 809 1800 Freephone: 1800 24 02 02 (ROI only) 0800 328 5875 (NI only)  
Fax: +353 (0)1 829 3966 Email: [healthie@airliquide.ie](mailto:healthie@airliquide.ie) Website: [www.airliquidehealthcare.ie](http://www.airliquidehealthcare.ie)